





HumidX™
F20



Rx Only

User Guide

English | Español de América | Deutsch | Français | Italiano
Nederlands | Português | Dansk | Suomi | Norsk | Español de España
Svenska | Český | 简体中文 | 繁體中文 | Bahasa Melayu | العربية

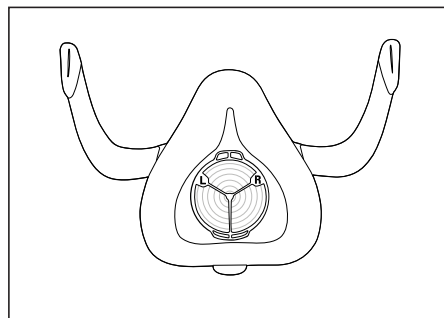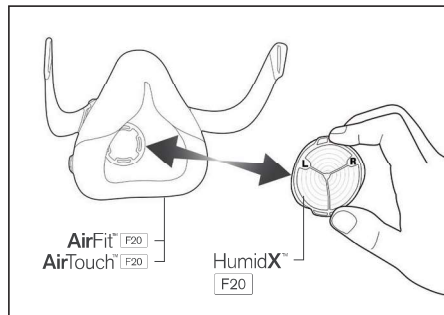

2

ENGLISH

The HumidX™ F20 is a waterless humidifier accessory, designed to provide more comfort by improving the moisture level in the air you breathe and offering relief from possible dryness.

The HumidX F20 is validated for use with the AirFit™ F20 and AirTouch™ F20 masks in conjunction with the AirMini™ device. It is intended for single-patient reuse in the home and hospital/institutional environment.

Using the HumidX F20

To add or remove the HumidX F20 refer to the illustration at the front of the guide.

Assembly

With the ResMed logo facing away, insert the HumidX F20 into the air inlet of the F20 mask until it clicks in place.

Disassembly

Remove the HumidX F20 from the mask.

Maintenance

The HumidX F20 must be replaced within 30 days of opening the sealed pack. When not in use, store in a clean and dry location.

The HumidX F20 cannot be washed as this will affect the performance of therapy. Remove the humidifier before cleaning your mask. For mask cleaning and reprocessing instructions, refer to your mask user guide.

Check the HumidX F20 daily for any signs of damage or blockages caused by dirt or dust and replace when necessary.

Disposal

The accessory does not contain any hazardous substances and may be disposed of with your normal household refuse.

Further information

For further information including troubleshooting, refer to AirMini user guide.

WARNING

- Remove all packaging before use.
- The accessory is not intended to be used simultaneously with medical-grade oxygen.
- Discontinue use of the accessory if you have ANY adverse reaction and consult your physician or sleep therapist.
- Only use compatible CPAP or bilevel therapy devices. Use in combination with

incompatible devices can decrease the safety or performance of the mask.

- Regularly inspect the accessory and follow maintenance instructions to prevent the growth of germs that can adversely affect your health.
- The accessory is not intended to be used with nebulizer medications that are in the air path of the mask.
- As with all masks, some rebreathing may occur at low pressures.

CAUTION

If any visible deterioration of the accessory is apparent (cracking, crazing, tears etc.), the accessory should be discarded and replaced.

Note: Any serious incidents that occur in relation to this accessory should be reported to ResMed and to the competent authority in your country.

Technical specifications

Environmental conditions

Operating temperature: 41°F to 104°F (5°C to 40°C)

Operating humidity: 15% to 95% RH non-condensing

Operating altitude: Sea level to 8,500' (2,591 m); air pressure range 1013 hPa to 738 hPa

Storage and transport temperature: -4°F to 140°F (-20°C to +60°C)

Storage and transport humidity: up to 95% RH non-condensing

Design Life

HumidX F20: 30 days

Symbols

 Importer  Medical device  Caution, consult accompanying documents  Replace 30 days after opening

See symbols glossary at [ResMed.com/symbols](https://www.resmed.com/symbols).

**ResMed Pty Ltd**

1 Elizabeth Macarthur Drive
Bella Vista NSW 2153 Australia

MANUFACTURER

See [ResMed.com](https://www.resmed.com) for other ResMed locations worldwide.
AirFit, AirTouch, AirMini and HumidX are trademarks and/or
registered trademarks of the ResMed family of companies.
For patent and other intellectual property information, see
[ResMed.com/ip](https://www.resmed.com/ip). © 2020 ResMed. 638217/2 2020-06

[ResMed.com](https://www.resmed.com)

